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EN: Instructions for use

CONCARE® CLASSIC conditioner's high wetting properties prolong comfort for contact lens wearers.

INSTRUCTIONS FOR USE:

1. Wash your hands.
2. After cleaning with **CONCARE® CLASSIC cleaner**, fill the lens case with **CONCARE® CLASSIC conditioner** and leave the lenses in the lens case overnight or for at least 6 hours. The contact lenses should be totally covered by the solution.
3. Remove the lenses from the lens case and rinse them briefly with **CONCARE® CLASSIC conditioner** before putting them on.
4. If you need to take the lenses out during the day, wet them with few drops of **CONCARE® CLASSIC conditioner** before putting them back on.

⚠ PRECAUTIONS:

- Always use a fresh solution to disinfect the contact lenses and dispose of any solution already used.
- Close the bottle tightly after each use.
- Do not touch the dropper with your fingers.
- Check that the seal is intact before using the product for the first time.
- Use the product within 90 days after opening.
- Keep out of the reach and sight of children.
- Do not use this product if you are allergic to any of its ingredients.
- In the event of eye irritation, remove the lenses, stop using the product and see a doctor or a contact lens specialist immediately.
- The lenses can be stored in the closed lens case for up to 30 days.

INGREDIENTS:

Sodium chloride, boric acid, sodium tetraborate, hydroxypropyl methylcellulose, poloxamer, 0.1 % disodium edetate and 0.0004 % polyhexamethylene biguanide in purified water

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
Conditioner solution

For disinfecting and preserving gas-permeable and rigid contact lenses

CE 0318

 Disop
Avda. Valelparra, 31-A
28108 Madrid, España

Vertrieb/ Distributeur/ Distributore/
Distributor/ Distribuidor/
Distributie:
Hecht Contactlinsen GmbH
Dorfstraße 2-4
D-79280 Au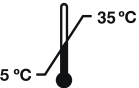

STERILE A