CONCARE® SEPT intense

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Peroxide System
for all types of contact lenses

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gb INNOVATION!
AVIZOR has developed a new contact lens cleaning product that has enhanced the possibilities and increased the benefits offered by existing solutions. Daily use of CONCARE® SEPT intense keeps lenses perfectly clean, making you feel like you are wearing new lenses every day. With this new product, lenses can be used longer and more comfortably. CONCARE® SEPT intense is the effective answer for contact lens users who seek maximum comfort and the highest standard of safety for their eyes. CONCARE® SEPT intense is a system that cleans, disinfects and thoroughly eliminates proteins. It is designed for ALL TYPES OF CONTACT LENSES*. No preservatives are used and no rubbing is necessary. The CONCARE® SEPT intense system tablets have been designed using the SELF ACTION™ formula that provides extraordinary results in cleaning lenses while maintaining them in perfect condition. Cleaning is so thorough that proteins are totally eliminated and cleaning with enzyme tablets is no longer necessary. CONCARE® SEPT intense can handle even the most hard-to-clean contact lenses, such as silicone and Saturn-type lenses. * Conventional soft contact lenses, rigid and gas-permeable lenses, disposable lenses and continuous wear lenses. Coloured lenses: in some cases, CONCARE® SEPT intense can alter lens colour and therefore the use of Avizor Multi-Purpose Solution is recommended for coloured lenses.

INSTRUCTIONS FOR USE:

1. Wash and dry your hands thoroughly before handling your lenses.
 2. Place the lenses in the baskets of the lens case.
 3. Fill the lens case cup to the line with CONCARE® SEPT intense disinfectant solution.
 4. Add one CONCARE® SEPT intense tablet to the cup containing the disinfectant solution.
 5. Place the lens case lid and close the case.
 6. Gently shake the lens case up and down to ensure the inside is disinfected.
 7. Make sure the tablet has fallen into the solution again so that neutralisation can begin. Store the case in a vertical position.
 8. The tablet will immediately begin to produce bubbles, and when neutralisation is complete (2 hours), the solution will be light green and your lenses will be disinfected, clean, neutralised and ready to use.
 9. Shake the lens case gently before removing the lenses.
 10. After removing the lenses, empty the case, rinse it with Avizor Saline solution and let it air dry.
- Last revision date: 2009/11

PRECAUTIONS:

- Follow usage instructions.
- Always use the lens case designed for CONCARE® SEPT intense. Other cases do not guarantee the correct results.
- Leave the lenses immersed for a minimum of 2 hours during the disinfecting and neutralisation process.
- If you observe that the solution in the lens case is colourless, you must repeat the lens disinfection process before using your lenses again.
- CONCARE® SEPT intense does not contain preservatives, and therefore we do not recommend you leave your lenses immersed in the solution for more than 48 hours. If you want to store lenses for a longer period, use Avizor Saline Solution.
- Do not use the product if the safety seal is missing or broken.
- Maintain the bottle well closed after use.
- Discard the solution in the lenscase after each use.
- Never use tap water to rinse your lenses or lens case.
- In case of irritation, stop using the lenses and consult your contact lens specialist.
- Do not use if you are allergic to any of the system components.
- Do not remove tablets from the package until you are ready to use them.
- Once opened, the bottle should not be used after 60 days.
- Store the disinfectant solution and the tablets at room temperature.
- Do not use after the expiration date.
- Do not ingest the solution or the tablets.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not put unneutralised CONCARE® SEPT intense disinfectant solution directly in your eyes.
- Keep out of children's reach.